
EcoLotion Transmission and Massage Lotion (with Aloe)

Manufacturer:

Eco-Med Pharmaceutical Inc.
151 Carlingview Drive, Unit 14
Etobicoke, Ontario M9W 5S4
www.eco-med.com

EC IREP Authorized EU Representative

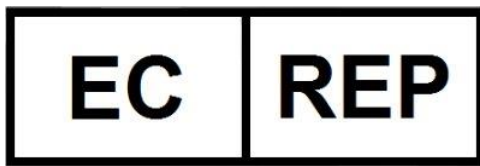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Description

EcoLotion Transmission and Massage Lotion is a non-sterile, white conductive medium that is specifically formulated for those who prefer a soothing formula. It is non-sensitizing and ideal for therapeutic procedures such as Shockwave Therapy and other massage therapies.

Indications for Use

Apply lotion directly to the diagnostic area. When the procedure has completed, the excess lotion can be removed with a soft paper towel or tissue or massaged into the skin. Clean equipment immediately after using.

Use open containers of ultrasound lotion is only for low risk procedures on intact skin and low risk patients.

Contraindications

Not for use where:

- Any invasive procedures in which a device is passed through the tissue (e.g., needle aspiration, needle localization, tissue biopsy, or other surgical procedures).

February 10, 2021

- Any procedure where the ultrasound lotion will have direct contact with breached or compromised skin, such as a fresh surgical site or an open wound.
- Any procedure where the ultrasound lotion may contact or come near mucosal membranes.

Warnings

- Do not use past labelled expiry date.
- If irritation occurs, discontinue use, and consult a physician.
- Use and dispose of as per regional requirements.
- Since there is no effective way to determine whether an opened bottle of lotion has been contaminated general guidelines are that lotion should be marked to expire 28 days after opening.

Precautions

- Do not use product if package is damaged or opened.
- To avoid contamination, tips of containers should not come into contact with patient, staff, or instruments.
- Use the product only in accordance with its intended use, see INDICATIONS FOR USE.

Recommended Storage Conditions

- To preserve product performance, do not freeze. If product is frozen, allow the product to naturally return to room temperature before use or heating and shake/mix well.
- Store in a dry area at room temperature (15°-30°C or 59°-86°F).

Reconditioning/Refilling

- Expiration dates on bulk containers need to be carefully monitored and containers discarded by the expiration date.
- Prior to refilling reusable bottles, ensure that the large bulk container of ultrasound lotion has not passed the expiration date.
- Prior to refilling, bottles must also be emptied and dried. Caked lotion must be thoroughly removed from the neck of the bottle and inside the bottle lid.
- Bottles must then be washed in hot soapy water and thoroughly rinsed. Only completely intact bottles may be reused.
- Bottles must be filled using a dispensing device that has been attached to the large bulk container and not by inserting the tip of the reusable bottle into the large bulk container to aspirate the contents.
- When refilling the bottle, do not touch the opening of either container with your hands or allow your hands to touch other environmental surfaces. Maintain aseptic technique during the refilling process.
- Bottles must be refilled as close as possible to the time of use.
- When in use, bottles must not be "topped off". They must be used until empty and then reprocessed prior to refilling.
- Dispensing nozzles must not come into direct contact with patients, staff, instrumentation, or the environment. If this is not possible, wipe the dispensing nozzle with a clean alcohol swab between patients. Wipe the outside of the ultrasound lotion bottle with a disinfecting wipe between patients.

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