# EcoGel 100 Ultrasound Imaging Gel

---

## Manufacturer:

Eco-Med Pharmaceutical Inc.  
151 Carlingview Drive, Unit 14  
Etobicoke, Ontario M9W 5S4  
[www.eco-med.com](http://www.eco-med.com)

## EC|REP Authorized EU Representative

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## Description

EcoGel 100 Imaging Ultrasound Gel is a non-sterile, low viscosity, green ultrasound gel formulated primarily for use in diagnostic imaging and other external procedures requiring a thinner medium. It is non-sensitizing and water soluble.

## Indications for Use

Apply gel directly to the diagnostic area. When the procedure has completed, the excess gel should be removed with a soft paper towel or tissue. Clean equipment immediately after using.

Use open containers of ultrasound gel only for low-risk procedures on intact skin and low risk patients.

## Contraindications

Not for use where:

- Any invasive procedures in which a device is passed through the tissue (e.g., needle aspiration, needle localization, tissue biopsy, or other surgical procedures).
- Any procedure where the ultrasound gel will have direct contact with breached or compromised skin, such as a fresh surgical site or an open wound.
- Any procedure where the ultrasound gel may contact or come near mucosal membranes.

### Warnings

- Do not use past labelled expiry date
- If irritation occurs, discontinue use and consult a physician
- Use and dispose of as per regional requirements
- Since there is no effective way to determine whether an opened bottle of gel has been contaminated general guidelines are that gel should be marked to expire 28 days after opening.

### Precautions

- Do not use product if package is damaged or opened.
- To avoid contamination, tips of containers should not come into contact with patient, staff or instruments.
- Use the product only in accordance with its intended use, see INDICATIONS FOR USE

### Warming of the Gel

- EcoGel 100 can be used at room temperature or warmed. If you choose to warm the gel prior to use, please refer to your health system's internal policies and procedures for gel warming temperature guidelines.

### Recommended Storage Conditions

- To preserve product performance, do not freeze. If product is frozen, allow the product to naturally return to room temperature before use or heating.
- Store in a dry area at room temperature (5°-40°C or 41°-104°F).

### Reconditioning/Refilling

- Expiration dates on bulk containers need to be carefully monitored and containers discarded by the expiration date.
- Prior to refilling reusable bottles, ensure that the large bulk container of ultrasound gel has not passed the expiration date.
- Prior to refilling, bottles must also be emptied and dried. Caked gel must be thoroughly removed from the neck of the bottle and inside the bottle lid.
- Bottles must then be washed in hot soapy water and thoroughly rinsed. Only completely intact bottles may be reused.

- Bottles must be filled using a dispensing device that has been attached to the large bulk container and not by inserting the tip of the reusable bottle into the large bulk container to aspirate the contents.
- When refilling the bottle, do not touch the opening of either container with your hands or allow your hands to touch other environmental surfaces. Maintain aseptic technique during the refilling process.
- Bottles must be refilled as close as possible to the time of use.
- Gel dispensed into a refillable bottle will expire in 30 days. The date of expiration must be written on the bottle at the time it is refilled.
- When in use, bottles must not be “topped off”. They must be used until empty and then reprocessed prior to refilling.
- Unused gel in a bottle or a refillable bottle must be discarded at the end of 30 days or by the manufacturer’s expiration date, whichever comes first.
- Dispensing nozzles must not come into direct contact with patients, staff, instrumentation, or the environment. If this is not possible, wipe the dispensing nozzle with a clean alcohol swab between patients. Wipe the outside of the ultrasound gel bottle with a disinfecting wipe between patients.

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